



# DuraGraft®

The World's 1<sup>st</sup> Endothelial Damage Inhibitor


## EASY PREPARATION\*

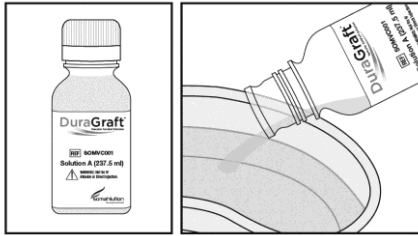

Pour the contents of **Solution A** into the container (tray, cup or small basin) where the vascular conduit will be stored prior to grafting.

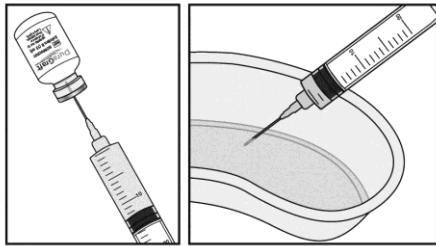


With a sterile syringe aseptically remove **12.5 ml of Solution B** and add it to the container with Solution A.

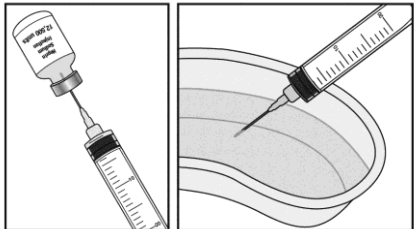

Add **12,500 units of heparin** to the container containing Solutions A and B.

**\*Heparin NOT provided in kit. Use heparin available in the surgical suite**

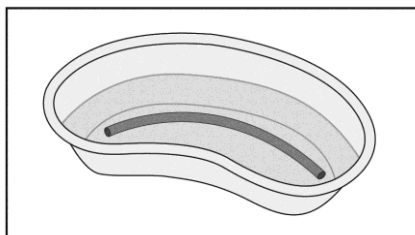

Mix by gently by swirling the container.

\*See IFU for more details

**Somahlution**

225 Chimney Corner Lane, Suite 2001, Jupiter, FL 33458

[www.somahlution.com](http://www.somahlution.com)

MKT-003 Rev 002